Distributed by:
berkeleyhealth
 52 Berkeley Square
 Mayfair London W1J 5BT UK
 +44 20 8191 0500
 support@berkeleyhealth.com
 www.berkeleyhealth.com

CE **REP** Qarad EC-REP BV
 Pas 257
 2440 Geel - Belgium

PRIMA LAB SA
 Via Antonio Monti 7
 CH-6828 Balerna - SWITZERLAND

+41 91 605 1030
 support@primallabs.ch
 primallabs.ch

REF 890021

890021-1_IFU_72_5.0 08/2022

Registered trademarks.

berkeleyhealth THYROID TSH

THYROID PREVENTION

- Self-test for the determination of the levels of thyroid-stimulating hormone (TSH) in whole blood samples**
 - Selbsttest zur Bestimmung des Thyreoidea-stimulierenden Hormons (TSH) in Vollblutproben**
 - Autotest pour la détermination des niveaux d'hormone stimulante de la thyroïde (TSH) dans les échantillons de sang entier**
- اختبار ذاتي لتحديد الهرمون المحفز للغدة الدرقية في عينات الدم الكامل

INSTRUCTIONS FOR USE GEBRAUCHSANWEISUNG NOTICE D'UTILISATION نشرة تعليمات الاستخدام

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Phoenix Innovative Healthcare Manufacturers Pvt. Ltd.
 FL-208, Shi Mahape Road,
 Electronic Zone, MIDC,
 TC Industrial Area, Mahape,
 Navi Mumbai 400 710 MH | India
 2197

EC **REP** Advena Ltd.,
 2nd Flr,
 Tower Street,
 Swatar, BKR 4013, Malta

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Hualan Tianda Medical Instruments Co., Ltd.
 No. 108, East Songjiang Road, Huayin
 Economic & Technological Development Zone
 223002 Hualan City, Jiangsu - China
 0197

CE **REP** Shanghai International
 Holding Corp GmbH
 (Europe), Effenstrasse 80,
 20537 Hamburg, Germany

SYMBOLS / SYMBOLE / SYMBOLES / الرموز

IVD In vitro diagnostic device In-Vitro-Diagnostikum Dispositif médical de diagnostic in vitro جهاز تشخيص مخبري	V Sufficient for <n> tests Ausreichend für <n> Tests Suffisant pour <n> tests كاف لعدد <n> اختبار	REF Lot number Katalognummer Code produit رقم القالب
I Read the instructions before use Beachten Sie die Gebrauchsanweisung Consulter le mode d'emploi اقرأ الإرشادات جيدا قبل الاستخدام	Tem Temperature limits Temperaturbegrenzung Limites de température حدود درجة الحرارة	LOT Lot number Chargenbezeichnung Code du lot رقم اللوط
STERILE R Sterilised using irradiation Sterilisation durch Bestrahlung Méthode de stérilisation utilisant l'irradiation مُعقم بالإشعاع	Do not reuse Nicht wiederverwenden Ne pas réutiliser يمنع إعادة الاستخدام	CE CE marking CE-Kennzeichnung Marquage CE علامة
EC REP Authorised Representative in the European Community Bevollmächtigter in der Europäischen Gemeinschaft Mandataire dans la Communauté européenne ممثل مُعتمد في المجتمع الأوروبي	Exp Expiry date (last day of the month) Verwendbar bis (letzter Tag des Monats) Ultimeur jusqu'à (dernier jour du mois) تاريخ انتهاء الصلاحية (آخر يوم من الشهر)	Legal manufacturer Hersteller Fabricant جهة التصنيع القانونية

ENGLISH TSH AND HYPOTHYROIDISM

The thyroid-stimulating hormone (TSH, also known as thyrotropin or thyrotropic hormone) is secreted by the adenohypophysis and controls the thyroid's secretion activity. When stimulated by the TSH, the thyroid produces and releases thyroxine (T4) and triiodothyronine (T3), which regulate the consumption and storage of energy in the body. When the levels of thyroid hormone in the blood are lower than normal, the adenohypophysis reacts and raises TSH levels to stimulate the thyroid. An increase in the concentration of TSH in the blood is usually an indication of reduced thyroid activity, which is a pathological condition better known as hypothyroidism. Hypothyroidism is the most common thyroid disorder. It affects men and women (with greater incidence among women) and it is a pathological condition characterised by a genetic predisposition. The symptoms of hypothyroidism include: tiredness, increased body weight, dry skin, depression, constant feeling of constipation and irregular periods in women. THYROID TSH TEST is a quick self-test that measures the level of TSH in the blood.

PRINCIPLE OF THE TEST

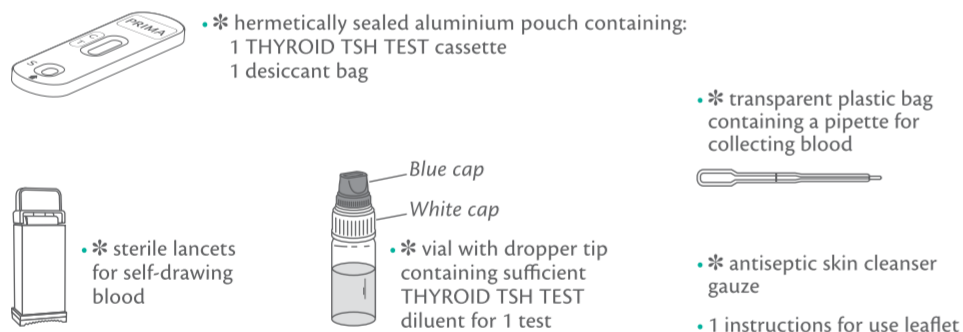
THYROID TSH TEST is an immunochromatographic test that detects TSH levels through special gold-conjugated monoclonal antibodies included in the reactive strip.

REFERENCES

- Yasaman Pirahanchi1; Ishwarlal Jialal2. Physiology, Thyroid Stimulating Hormone (TSH). StatPearls Publishing; 2018 Jan.-
- Simon H.S. Pearce,a,b,* Georg Brabant,c Leonidas H. Duntas,d Fabio Monzani,e Robin P. Peeters,f Salman Razvi,a,gand Jean-Louis Wemeauh " 2013 ETA Guideline: Management of Subclinical Hypothyroidism". Eur Thyroid J. 2013 Dec; 2(4): 215-228.
- De Groot L, Abalovich M, Alexander EK, Amino N, Barbour L, Cobin RH, Eastman CJ, Lazarus JH, Luton D, Mandel SJ et al. Management of thyroid dysfunction during pregnancy and postpartum: an Endocrine Society clinical practice guideline. Journal of Clinical Endocrinology and Metabolism 2012 97 2543–2565. (doi:10.1210/jc.2011-2803)
- Nat Rev Endocrinol. 2018 May;14(5):301-316. doi: 10.1038/nrendo.2018.18. Epub 2018 Mar 23."Global epidemiology of hyperthyroidism and hypothyroidism."Taylor PN1, Albrecht D2, Scholz A1, Gutierrez-Buey G3, Lazarus JH1, Dayan CM1, Okosieme OE1.

CONTENT

* The number of the testing devices of the kit may vary. For the exact number of tests contained, please refer to the "content" section on the external box.

- * Material required but not supplied: cotton wool, a device to measure time (i.e. timer, watch).
- * Do not open the sealed aluminium bag until just before performing the test. Take care to open it as marked.
- * The desiccant packet must not be used. Dispose of it with household waste without opening it.

PRECAUTIONS

- Read these instructions for use carefully before performing the test. The Test is reliable only if all the instructions are followed correctly.
- Keep the Test out of the reach of children.
- Do not use the Test after the expiry date or if the package has been damaged.
- Follow the procedure exactly, using only the specified quantities of blood and diluent.
- Store the Test components at a temperature between +4°C and +30°C. Do not freeze.
- Use the test and lancing device once only.
- The test is for external use only. DO NOT SWALLOW.
- In vitro diagnostic device for individual use.
- Not recommended for people who take anti-coagulant medications (blood thinners) or people suffering from haemophilia.
- After using, please dispose of all components according to your local waste disposal laws. Ask your pharmacist for advice.

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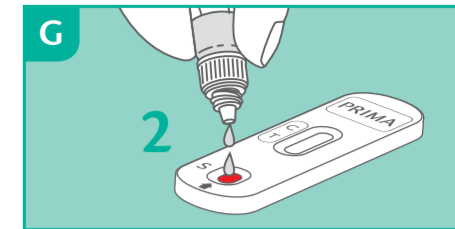
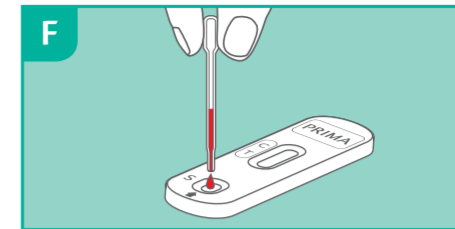
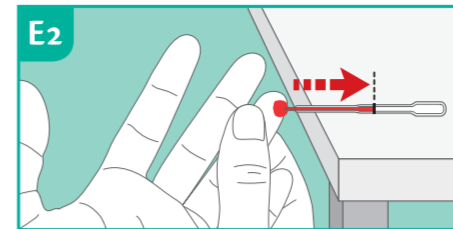
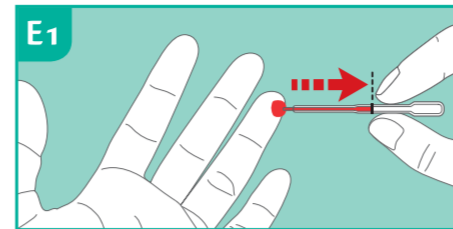
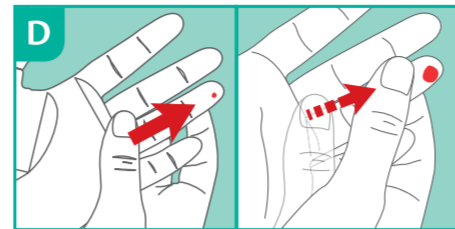
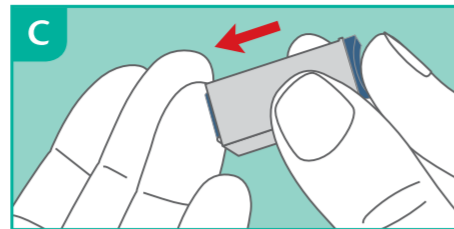
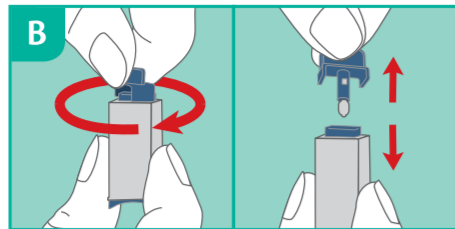
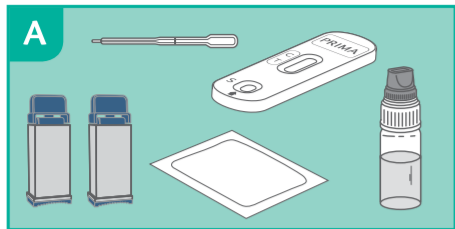
ENGLISH TEST PROCEDURE

- 1) Wash hands with soap and warm water, rinse with clean water and allow to dry. If this is not possible, use the provided gauze as an alternative.
2) Prepare the necessary material as follows: open the aluminium pouch, take out only the test cassette and throw away the desiccant bag.
3) Carefully rotate the protective cap of the sterile lancet 360° without pulling it.
4) Carefully massage the finger chosen for the puncture (the side of the ring finger is recommended).
5) Holding the hand down, massage the finger until a large drop of blood forms.
6) Take the pipette without pressing the bulb.
7) Place the blood collected with the pipette into the well indicated on the cassette (S) by pressing the pipette bulb.
8) Unscrew the blue cap from the dropper vial (leave the white cap tightly screwed on).
9) Wait 10 minutes and read the results referring to the next section for result interpretation.

RESULTS INTERPRETATION

READ THE RESULTS AFTER 10 MINUTES (NOT OVER 15 MINUTES)
The intensity of the line colours is not relevant for the purposes of interpretation of the Test's results.

POSITIVE RESULT

Two coloured bands appear in the reading window by the T (test) and C (control) signs. The T band may be less intense (lighter) than the C line. This result means that the levels of TSH in the blood are equal to or higher than 5 µIU/ml and therefore you should contact a physician.

NEGATIVE RESULTS

A coloured band appears only under the C (control) sign. This result means the level of TSH is normal.

NON VALID RESULT

No bands appear or there is a line only under the T (test) sign and not under the C (control) sign. In this case it is not possible to interpret the result of the test, which must be considered not valid. Repeat the test with a new blood sample.

FAQ - QUESTIONS AND ANSWERS

HOW DOES THE THYROID TSH TEST WORK?

The thyroid-stimulating hormone (TSH) controls the activity of the thyroid gland. If the test result is positive, the concentration of TSH is above 5 µIU/ml, so it is likely that thyroid gland activity is reduced.

WHEN CAN THE TEST BE USED? If hypothyroidism symptoms are present (such as tiredness, depression or feeling cold, weight gain, dry skin, fragile hair, persistent constipation or irregular periods in women), it is advisable to screen for thyroid activity.

HOW DO I INTERPRET THE TEST IF THE COLOUR AND INTENSITY OF THE TEST AND CONTROL BANDS ARE DIFFERENT? The colour and the intensity of the bands are not important for the interpretation of the result.

HOW DO I INTERPRET THE TEST IF THE COLOUR AND INTENSITY OF THE TEST AND CONTROL BANDS ARE DIFFERENT? The colour and the intensity of the bands are not important for the interpretation of the result.

IS THE RESULT RELIABLE IF READ AFTER 15 MINUTES? No. The test must be read 10 minutes after the procedure is completed and no later than 15 minutes from this time.

WHAT SHOULD I DO IF THE RESULT IS POSITIVE? If the result is positive, the concentration of TSH in the blood is above normal levels. Talk to your GP. A positive result in this test is not enough to diagnose hypothyroidism. Thyroid hormones must also be measured with TSH to distinguish primary hypothyroidism from a condition with altered TSH secretion.

HOW ACCURATE IS THE THYROID TSH TEST? The Test is very accurate. Assessment reports show a concordance rate above 95% with reference methods (CI 95%: 90.7 - 99.2).

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